

IVTM – Hospital Monitor Adapter

Streamline Data Management

In today's world of electronic patient recordkeeping, the Hospital Monitor Interface Accessory (HMIA) simplifies patient temperature monitoring when using the ZOLL® IVTM™ System to manage patient temperature. Patient core temperature monitored directly by the ZOLL IVTM System is sent to the patient monitoring system for display and the hospital's electronic patient record system for documentation.



IVTM System



Patient temperature displayed on both patient monitor and the IVTM System while using only one temperature probe.



Bedside Monitor



Connect All Patient Monitors with One Temperature Probe

The HMIA enables easy display of a patient's temperature on both the IVTM System and patient monitors using a single temperature probe. Compatible with the majority of patient monitoring systems, the HMIA interface cable plugs into the patient cable of the bedside monitor. Once connected, the bedside monitor processes the HMIA signal in the same way it does the signal from a patient temperature probe. The patient temperature can be displayed on the bedside and central station monitors, and if the patient monitoring system is networked to electronic patient records, the data is seamlessly transmitted to the record.


Connection is simple. It is as easy as plugging one cable into another. After that, nursing and medical staff have real-time access to all relevant patient temperature data – from whatever monitor is most convenient for them.

Enhance Electronic Patient Record Keeping with HMIA

- Streamline data collection
- A single temperature probe provides display on both monitoring and the IVTM Systems
- No separate controls – the HMIA interface module works automatically whenever the IVTM System is on
- Connects easily to patient monitor

HMIA Information Flow to Hospital Monitor

HMIA connects to monitor cable in identical fashion as a patient temperature probe.

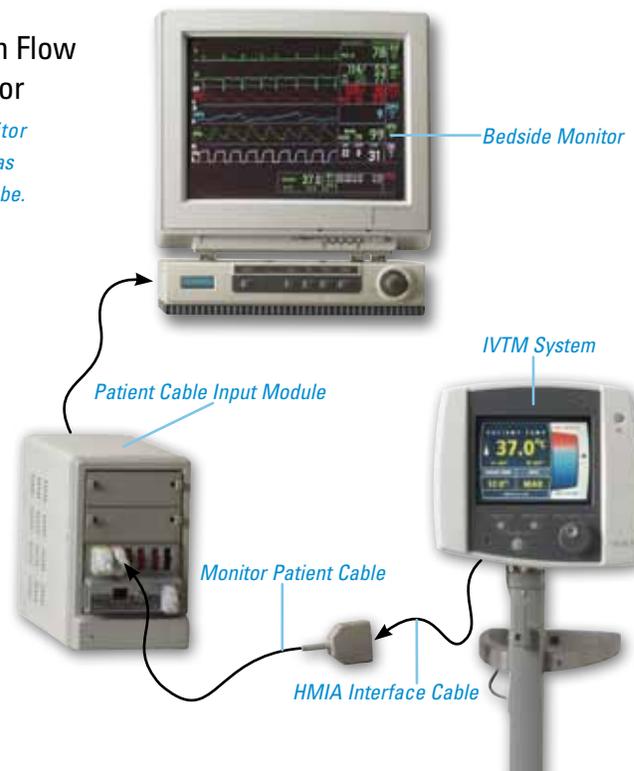

ZOLL Medical Corporation
269 Mill Road
Chelmsford, MA 01824
+1-978-421-9655
+1-800-348-9011
www.zoll.com

Printed in U.S.A.
061010
104138-001 Rev. 2
9656-0217

©2010 ZOLL Medical Corporation. IVTM and ZOLL are trademarks or registered trademarks of ZOLL Medical Corporation in the United States and/or other countries.